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MD Medical Device

LATEX

Caution: This device contains natural rubber latex which may cause allergic reactions.

INTENDED USE

The device is intended for compression around the knee and external support or stabilization of the patella

Indications for use

For knee conditions that may benefit from compression, such as:

- Mild sprains and strains
- Patella tendinitis
- Knee pain
- Feeling of instability
- Chondromalacia

Contraindications

Must not be used by individuals for whom compression is contraindicated.

Warnings and Cautions:

The device should be used directly on intact skin and never over an open wound.

• The device should not be used for extended periods of time and removed during long periods of rest, e.g., sleep.

• Regular healthcare professional supervision is recommended for patients with peripheral vascular disease, neuropathy, and sensitive skin

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

The patient should stop using the device and contact a healthcare professional:

• If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.

• If any pain, skin irritation, excessive pressure, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

USAGE

Size Selection

Measure the calf and thigh circumference 15 cm (6") above and below mid-patella. The sizing chart is shown on the packaging; if between sizes always select the smaller size for optimal fit.

Device Application

1. Fold the top part of the device outward so that the FoldLock edge becomes visible (Fig. 1). Grasp the device with two hands on the side and pull the device over the knee (Fig. 2).

2. Pull evenly over the knee joint until the SmartPad encloses the knee cap and the knee cap is free from pressure. If preferred by the user, the SmartPad can be removed (Fig. 3).

3. Fold the edge of the device back into the original position (Fig. 4).

To remove the metal stays turn the device inside out and remove the stays out of the stay pocket (Fig. 5).

Device Removal

Fold the FoldLock edge of the device outward so that the edge becomes visible. Grip the device at the level of the calf and pull it off in a downward direction.

Cleaning and care

• Machine-wash with a mild detergent at cold temperature (30°C).

• Air dry.

Note: Do not tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

Liability

Össur does not assume liability for the following:

• Device not maintained as instructed by the instructions for use.

• Device assembled with components from other manufacturers.

• Device used outside of recommended use condition, application, or environment.

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