READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT

Keep this manual in a safe location for future reference

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Content

1. IMPORTANT SAFETY INSTRUCTIONS.....	2	6. Troubleshooting Guide	5
2. Parts	4	7. Specifications.....	5
3. To Set Up.....	4	8. FCC Statement.....	6
4. To Use	4	9. Warranty	6
5. Care and Storage	5		

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

With best regards,
Your Beurer Team

Why use a daylight lamp?

When the hours of daylight decrease in autumn and people stay inside more, the effects of a lack of natural light may accumulate. Sunlight indirectly controls the production of melatonin, which is released into the blood only in darkness. This hormone tells the body when it is time to sleep. In months with less sunlight, production of melatonin increases, which makes it harder to get up in the morning.

Normal electric light is not sufficient to influence the hormonal balance, since even a well-lit office luminance level is only around 500 lux.

Read these instructions carefully and keep them for later use, and be sure to make them accessible to other users.

1. IMPORTANT SAFETY INSTRUCTIONS

Signs and symbols

The following signs appear in the Safety Section (page 2) and in this manual on page 3 and 4.

READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

	WARNING	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	CAUTION	CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
	IMPORTANT	Safety note regarding potential for damage to the device/accessories

NOTICE NOTICE addresses practices not related to personal injury, such as product and/or property damage.

Device in protection class 2

IP 21

Protected against foreign objects equal to or greater than 12.5 mm in diameter and against vertically falling drops of water

Follow Instructions for use

For home use only

Energy efficiency class

WARNING:

- The light of the daylight lamp is only intended for use on the human body.
- Before use, ensure all packaging materials are removed and that there is no visible damage to the unit or accessories. When in doubt, do not use the unit and contact customer service.
- Always place the daylight lamp on a flat, level, stable, hard surface.
- Do not immerse the unit into water and do not use it in wet rooms such as a bathroom, laundry room, or pool area.
- Do not use it in the vicinity of people who are helpless, children under age 3 or people who are insensitive to heat (people with skin lesions due to illness).
- Do not use after taking painkillers or drugs or drinking alcohol.
- Keep children away from packaging materials (risk of suffocation).
- Do not cover or store the unit while it is warm.
- Always unplug the power adapter and allow the device to cool down before touching it.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. Operate only when the unit is completely dry.
- Ensure that you only insert and remove the power adapter with dry hands and that you only press the ON/OFF button with dry hands.
- Keep the power cord away from heated objects and flames.
- Protect the unit from strong impacts.
- Do not disconnect the power adapter from the electrical outlet by pulling on the cord.
- This product has no user-serviceable parts; return the product to customer service for any repair or replacement.
- If you are taking medication such as pain relievers, anti-hypertensives or antidepressants, consult your doctor before using the daylight lamp.
- People with retinal diseases as well as diabetics need to be examined by an ophthalmologist before using this daylight lamp.
- Do not use if you have an eye disease such as cataracts, glaucoma, diseases of the optic nerve in general and inflammation of the vitreous humor.
- If you have health concerns of any kind, contact your doctor before using this product.
- Danger of strangulation due to excessively long cables.
- Disconnection from the power supply network is only guaranteed when the mains part is unplugged.
- No calibration and no preventive checks or maintenance are to be carried out on this device.
- Do not make any changes to the device without the manufacturer's permission.
- If the device has been changed, thorough tests and checks must be carried out to ensure the continued safety of further use of the device.
- Do not position the device in a location where it is difficult to disconnect it from the mains supply.
- Check the device before each use. Do not use the device if it is damaged in any way. The continuous use of a damaged unit can be dangerous and may cause injury or improper results.
- Light sources are excluded from the guarantee.

2. Parts

1. Fluorescent screen
2. Timer Adjustment Buttons
3. Timer LEDs
4. Bluetooth LED
5. Brightness Adjustment Buttons
6. On/Off Button
7. AC Adapter Socket

3. To Set Up

Remove plastic wrapping and check the device for damage. If you notice any damage, contact customer service.

1. Place the device on a flat, level, stable, hard surface so that it will be 8 – 24 inches (20 – 60 cm) away from you during use.
2. To prevent product damage, only use the supplied AC Power Adapter.
3. Insert the round power adapter plug into the AC Adapter Socket on the rear of the Daylight Lamp Stand. Connect the AC Power Adapter to a standard 110 VAC electrical outlet.

4. To Use

1. Press the On/Off Button. If you do not press any Timer Buttons, the light will stay on until you press the On/Off Button again. To use the Timer, see step 3.
2. Sit 8 – 24 inches (20 – 60 cm) from the lamp. You can go about your normal activities during use, such as reading, writing, using your phone, etc.
3. To use the Timer, press the +/- Timer Adjustment Buttons until the desired Timer LED illuminates on the front panel. Usage time can be set to 15, 30, 45, 90, or 120 minutes. To deactivate the timer function, press the “-” Button repeatedly until all LEDs shut off.
4. Usage duration depends on the distance; the closer you are to the light, the less time you should use it.

Distance	Duration
8 inches (20 cm)	½ hour per day
8 – 16 inches (20 – 40 cm)	1 hour per day
16 – 24 inches (40 – 60 cm)	2 hours per day

5. To set the brightness level, press the +/- Buttons on the top of the lamp to increase or decrease the brightness level. This lamp has 10 levels of brightness.
6. To activate the color-changing feature, when device is off, press and hold the On/Off Button for two seconds. Press the On/Off Button again while this feature is activated to keep it on indefinitely and keep it at current color. Press the On/Off Button again to deactivate.
7. Periodically look directly into the light very briefly. Do not stare directly at the light, however, since that could overstimulate the retina.
8. For more effective results use for at least seven consecutive days between 6 AM and 8 PM for two hours per day.
9. Start with shorter periods of usage and increase over the course of a week.
OTE: Eye aches and mild headaches may occur for the first few uses and will subside after continued use.
10. Press the On/Off Button to shut the unit off. Unplug the AC Power Adapter from the electrical outlet.

CAUTION

The light remains hot after use. Let the lamp cool before storing.

Beurer LightUp app

You can control all of the device's functions with the Beurer LightUp app on your smartphone. To control the device's functions with the app on your smartphone, proceed as follows:

1. Ensure the device is switched on.
2. Download the Beurer LightUp app from the Apple App Store (iOS) or Google Play Store (Android).
3. Open the Beurer LightUp app.
4. Activate Bluetooth® on your smartphone. In the Bluetooth menu on your smartphone, select TL 100; the blue Bluetooth LED on the device's front panel will illuminate when it is paired.
5. Select the TL 100 device in the app settings. You can now control the device from the app.

Light

You can switch the device on and off using the app as well as adjust brightness and set the timer.

- By clicking the Light Bulb image on your phone, you can turn the device on and off.
- Timer and brightness can also be controlled by moving their corresponding slides on the app.

Mood light

You can also use the app to adjust the mood light. You can select from ten pre-configured mood light settings (scenes, each with its own brightness control and timer). After activating a mood light setting, the color-changing mood light illuminates. The pre-configured mood light settings can only be adjusted from the app.

- Turn on the moodlight by clicking the power button in the "moodlight" section of your app.
- Here you can use the slider to select your desired brightness, scene, and duration of light.

5. Care and Storage

- Before cleaning the unit, always switch it off, unplug it, and let it cool.

Do not immerse unit in water or other liquids. Use a slightly damp cloth to clean the device. Do not use abrasive detergents.

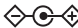
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
- If you are not going to use the device for an extended period of time, disconnect it and store it in a dry place, out of the reach of children.

6. Troubleshooting Guide

Problem	Possible cause	Solution
Device does not light up	On/Off Button switched off.	Press the On/Off Button.
	No electricity.	Power cord is faulty or not connected to a working outlet.
	LEDs have reached the end of their service life or are faulty.	Contact customer service for repair or replacement.

7. Specifications

Dimensions (W x H x D)	14.4 x 13.2 x 8 inches (366 x 335 x 203 mm)
Weight	40 oz. (1130 g) [43.8 oz. (1243 g) including AC Power Adapter]
Lighting elements	LED
Power	36 Watts
Light intensity	approx. 10,000 lux (@10 cm)
Radiation	Output of radiation beyond the visible spectrum (infrared and UV) is so low that it is harmless to eyes and skin.

Operating conditions	41° – 95 °F (5° – 35 °C), 15 – 90 % relative humidity, 86 – 106 kpa atmospheric pressure
Storage conditions	-4° – 122 °F (-20° – 50 °C), 15 – 90 % relative humidity, 86 – 106 kpa atmospheric pressure
Product classification	External power supply, Protection class II, IP21
Color temperature	5,300° K
Power Adapter Input	100 – 240 VAC ~50/60 Hz
Power Adapter Output	24 VDC, 1.5 A
Polarity	 Center Positive

8. FCC Statement

Changes or modifications to the product not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with Industry Canada Licence-exempt RSSs. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

9. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer daylight lamp, Model TL 100, excluding the battery, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer daylight lamp, Model TL 100, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part

as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: www.beurer.com

Distributed by:
Beurer North America LP
900 N Federal Highway, Suite 300
Hallandale Beach, FL 33009
www.beurer.com

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