

Model No: 10211C

Maximum User Weight: 120kg

Coopers Rolling Shower and Commode chairs have the exclusive purpose of providing a means in which a person can have their bathing and or toileting needs addressed. It can be used either as part of a rehabilitation programme e.g. when the user is recovering from an injury or operation; and as a long term bathing / toileting aid, e.g. when the user has permanent difficulty with walking and standing in a shower and or lowering or raising themselves fully into bath or onto a standard toilet. Coopers Rolling Shower and Commode chairs are for indoor use only



Before using this product, please read these instructions carefully. They contain instructions for the safe and proper use of your chair.

PROPER USE

The chair is to be used by a single occupant solely as an aid to showering or bathing, going to the toilet or for transfer indoors. Any other use is prohibited. Do not use the chair outdoors.

PARTS SUPPLIED

The chair is supplied with the following parts:

1. Backrest with armrests (upper frame)
2. Seat frame and base plate
3. Castor wheels (x4)
4. Footrests (x2)
5. Seat with cut-out
6. Commode pan with lid
7. Securing knobs (x2)
8. Horseshoe clips (x10)

SETTING UP

SET THE SEAT FRAME AND BASE PLATE

1. Lift the seat frame up slightly out of the base plate. Align the holes in the seat frame with the holes in the base plate and secure in place using a horseshoe clip.
2. Ensure that each corner of the frame is set at the same height.

ATTACH THE CASTOR WHEELS

1. Place the seat frame on its side.
2. Insert one of the castor wheels into the tubing at the corner of the frame.
3. Line up the holes in both tubes and secure in place using one of the horseshoe clips. Push the clip firmly until it audibly locks in place and sits flush with the tube.
4. Repeat this for the other corner.
5. Turn the frame over and insert the remaining two wheels as above.

ATTACH THE SEAT

1. Lock all four wheels by pressing the four brakes with your foot.
2. Attach the seat by clipping it onto the seat frame and pushing down firmly. Ensure all four clips are attached to the seat frame. The cut out section of the seat should be positioned at the front of the chair.

ATTACH THE BACKREST

1. Take the backrest section and standing at the rear of the chair fit into the base frame.
2. Secure by inserting and tightening the black securing knobs.

ATTACH THE FOOTRESTS

1. Insert the footrest tube into the open tube at the front of the frame. Secure by pushing a horseshoe clip through the hole in the footrest and frame tube.
2. Repeat with the other footrest. Ensure that both footrests are set at the same height.

ATTACH THE COMMODOE PAN

1. Slide the commode pan under the seat from the rear of the chair.

HEIGHT ADJUSTMENT

ADJUSTING THE SEAT HEIGHT

The seat height can be adjusted to suit the user. As a rule the seat is at the correct height when the user is sitting on the seat and both feet touch the ground.

The height must be the same on both sides.

Do not attempt to adjust the height with the user sitting in it.

1. Remove the commode pan, seat and backrest by reversing the instructions previously given.
2. Remove the horseshoe clips securing the seat frame to the base plate.
3. Adjust the height of the seat frame, then secure by re-inserting the horseshoe clips.

ADJUSTING THE HEIGHT OF THE FOOTRESTS

1. Remove the horseshoe clips.
2. Set the footrest at the correct height and line up the holes.
3. Re-insert the horseshoe clips.

INSTRUCTIONS FOR SAFE USE

1. Always apply the brakes before transferring a person onto or from the chair and when the chair is stationary.
2. Do not exceed the stated max user weight.
3. For indoor use only on flat hard surfaces.
4. Do not pick up the chair whilst the user is seated.
5. Remove the footplates before transfer.
6. For side transfer, lift up the armrest.
7. After transfer, replace the armrest into position ensuring that it has locked firmly in place.

CAUTION! When folding the armrests up or down, take care not to trap any body parts

8. Replace the footrests and lock into position using the clips. Do not stand on the footrests
9. Always make sure that the commode pan has been securely locked into place before use.
10. Do not use or store in extreme temperatures and avoid storing in direct sunlight.

CLEANING & MAINTENANCE

- Regularly check and tighten as required all clips, nuts and bolts.
- Regularly inspect the entire chair for fatigue and replace parts where necessary including brakes.
- It is recommended that the chair is cleaned regularly with a soft cloth and soapy water. Dry with a soft cloth.
- Do not use any abrasive cleaners or wire wool pads. The commode pan cannot be cleaned in an autoclave.
- Always thoroughly inspect the chair before use after maintenance or cleaning to ensure that it is functioning correctly.

DISPOSAL

The product does not contain any hazardous substances. Dispose of the individual parts at your local facility.

WARRANTY

All Coopers products are warranted for 12 months from the date of purchase against manufacturing or design defects.

The warranty is subject to the following conditions:

- It does not apply to wear and tear, inappropriate or incorrect use, or non-observance of User Instructions.
- Defective set up by the user or third parties or negligent or careless treatment.
- Use of unsuitable or unapproved spares or unauthorised alteration.
- The warranty does not extend to consequential costs resulting from fault clearance, loss of earnings or expenses

THIS IN NO WAY AFFECTS YOUR STATUTORY RIGHTS.

LIFE EXPECTANCY

We estimate a life expectancy of 2 years for this product, provided that:

- It is used in strict accordance with the intended use as set out in this document.
- All service and maintenance requirements are met.

The estimated life expectancy can be exceeded if the product is carefully used and properly maintained. The life expectancy can also be considerably reduced by extreme or incorrect usage. The fact that we estimate a life expectancy for this product does not constitute an additional warranty.



Sunrise Medical Limited

Thorns Road, Brierley Hill, West Midlands, DY5 2LD England
Tel + 44 (0) 845 605 6688 Fax + 44 (0) 845 605 6689



Conforms to
The Medical Device Directive 93/42/EEC 1993