



The bed backrest is suitable for use in bed to tilt your back to aid with reading, watching television, etc. For your comfort there are a number of angle settings for the backrest and a padded headrest. The backrest also folds up for storage.

The backrest should be placed and used on top of the mattress and bed sheets. It is suitable for one user only. The headrest is secured on the backrest by velcro. The headrest can be removed from the backrest if not required. To remove the headrest pull up on the headrest until the velcro is released. Then store the headrest behind the backrest.

Please note – the headrest can be removed from the front area of the backrest but still remains attached to the device by a fabric strap. Do not remove the headrest from the device completely.

Assembly Instructions:

Lay the backrest out flat then pull up the top of the backrest (as shown in the photograph below). When the backrest is at the angle you

require, push the cross bar down into the nearest plastic groove on the bottom of the backrest. Ensure the backrest is securely held in position before use.



To fold the backrest:

Pull up on the top of the backrest. Remove the cross bar from the groove and fold up in the backrest. Fold the backrest down flat.

Warranty:

The backrest is covered by a twelve-month warranty from date of purchase. It covers design and manufacturing defects. It does not cover upholstery, wear and tear or accidental damage. Please see your dealer for full details of the warranty.

Cleaning Instructions

The bed backrest can be wiped down with a damp cloth and mild detergent.

Manufacturer:

Drive DeVilbiss Healthcare Ltd,
Heathfield Lane, Birkenshw, BD11 2HW.
Web: www.drivedevilbiss.co.uk
Email: enquiries@drive-devilbiss.co.uk