

## Warranty details

Your Drive Devilbiss branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase. This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your local Drive stockist.

### **Drive Devilbiss Healthcare Ltd**

Sidhil Business Park, Holmfield, Halifax,  
West Yorkshire, HX2 9TN, Great Britain

info@drivedevilbiss.co.uk  
www.drivedevilbiss.co.uk

Tel: +44 (0) 845 0600 333

### **Drive Devilbiss Sidhil Ltd**

4 Trench Road, Mallusk, Newtownabbey,  
BT36 4TY, Northern Ireland

**drive**

**DeVilbiss**  
HEALTHCARE



## Elevated Toilet Seat with Removable Arms

### Instruction Manual

Please read these instructions carefully before assembling or using this product. Ensure you are familiar with all of the safety and operating instructions.

## Installation instructions

1. Remove product from carton and read all instructions carefully before attempting to install product.
2. To install arms, depress push pins on arm and insert into mounting tubes. There will be an audible click. Ensure the push pins fully protrude through the holes in the mounting tubes. To remove arms, reverse steps.
3. Lift toilet lid and seat assembly and place rear wings of toilet seat around rim of toilet bowl. (Some toilet bowls may require that seat assembly be removed to allow proper fit.) In addition the front adjustment bracket must clear the rim in order for the elevated toilet seat to fit securely.
4. With elevated toilet seat resting correctly on rim, straighten and centre product over toilet bowl as necessary. Turn adjusting knob clockwise to tighten product securely in place. Hand tighten only - be careful not to over-tighten, as threads on adjustment bracket may strip.
5. Check for secure fit routinely before using product. User's weight should be centred over toilet seat, not too far forward or too far to one side, as unit may tip.
6. If this product does not fit your type of toilet correctly, or cannot be tightened securely as noted above, it is recommended that you discontinue use.

## Removal instructions

1. Rotate knob counter clockwise until seat is loose.
2. Lift front of elevated toilet seat and pull forward to remove.

## Safety precautions

1. Always ensure that the seat is correctly and securely locked in place before using.
2. Ensure that the adjustment knob in front of the seat is tight at all times, thus securing the seat to the toilet bowl.
3. This product is designed for use with the most common types of toilets. It will not fit all toilet types; be sure that the unit is correctly installed, fits properly and is stable before use.
4. On models with arm rests: The arms are for the purpose of providing assistance when getting on and off the seat. **DO NOT** try to use the arms to support full body weight during transfer, as this may result in failure of the unit.

## Cleaning instructions

1. Use a non-abrasive detergent or household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product, as damage may occur.

## Specification

Product Code: 12027RA

Maximum user mass: 135kg (21st)

Seat Height: 13cm

Seat Width: 43cm

Seat Depth: 42cm