

INSTRUCTIONS FOR USE

⚠ WARNING: To minimise the risk of injury from improper and unsafe use of this grab bar, read these instructions and safety precautions before use.

Intended Purpose:

The grab bar is intended to assist users with manoeuvring and to aid balance.

Introduction:

The grab bar is ready for use on suitable flat and smooth surfaces (e.g. tiled surfaces). If you have questions or need additional information, contact Drive DeVilbiss Healthcare Ltd or your approved supplier.

For more information, or assistance with use and maintenance, please contact your provider. For the latest version of this document, contact Drive DeVilbiss Healthcare Ltd using the details below, or check our website.

SETUP

Before using the grab bar, thoroughly clean the suction cups, then clean and dry the surface being used.

Press the rubber discs firmly on the cleaned surface and press all levers down to a flat position. Ensure the discs do not lie over any gaps, objects, grouting lines or other joints.

Test the bar is secure before each use. If your grab bar has an indicator, it should be coloured **GREEN** once attached.

If the bar is not secure, or any of the indicators are **RED**, remove the bar, clean the surfaces and reattach. Do not use the grab bar until the required suction power has been achieved.



SAFETY PRECAUTIONS

This product is designed to provide support. It is not designed to hold a person's entire body weight. **Test the grab bar is stable before every use**, and always reattach if loose.

- Not intended for use on grouting lines or joints (see picture).
- Do not use unless individual parts are joined together securely, and the push button is fully engaged.
- **Stop using the grab bar immediately if the handle becomes loose**, or if the grab handle becomes damaged.



MAINTENANCE

- Check the handle is secure before every use.
Do not use if loose, worn or damaged.
- Check the suction cups are not worn or damaged.
- Clean the grab bar occasionally using water and a mild detergent.
Use the release tabs to remove for easy access when required.

FEATURES

- Portable grab bar
- Release levers for quick and easy install and removal
- Large suction cups provide a strong hold without marking property
- Easy to wipe clean

Comments, Feedback, or Questions

For assistance with using your grab bar or for any warranty, sales or customer service information, please contact your distributor or retailer. If in doubt, contact Drive DeVilbiss Healthcare Ltd.

SPECIFICATIONS

	Product Code	Handles	Length	Suction Cup Size	Indicator	Colour
Rotating Suction Cup Grab Bar (19¾")	RTL13084EU	Two, Rotating	19 ¾" 50 cm	3 ½" 9 cm	Yes	White/ orange
Suction Cup Grab Bar 12" with Indicator	13082-3	One	12" 30 cm	3 ½" 9 cm	Yes	White/ orange
	13082-2PACK	One	12" 30 cm	3 ½" 9 cm	Yes	White/ orange
Suction Cup 12" Grab Bar	GRSC12-2	One	12" 30 cm	3" 8 cm	No	White/ grey

REMOVAL

To remove the grab bar, simply pull the levers upwards (1), then pull the small tabs (2) to release each suction cup.

When the product has come to the end of its useful life, follow local recycling and disposal policies.



Drive DeVilbiss Healthcare Ltd.,

Sidhil Business Park, Holmfield, Halifax, West Yorkshire, HX2 9TN, Great Britain

info@drivedevilbiss.co.uk

www.drivedevilbiss.co.uk

Customer Service: +44 (0) 845 0600 333

Spares: +44 (0) 1274 475 125

Drive DeVilbiss Sidhil Ltd.,

4 Trench Road, Mallusk, Newtownabbey, BT36 4TY, Northern Ireland