# KINGFISHER<sup>TM</sup> Bath/Shower Board

How to assemble, fit and general maintenance














## How to Assemble

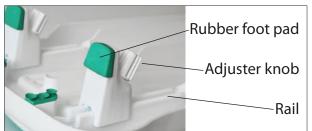

The bath board is width adjustable to fit securely against side of different width/shape baths



With the rubber foot pad facing outwards, slide the 'tongue' under the rail (pull gently up to ensure the foot is engaged under the rail as it should not lift off)



Loosen the adjuster knob on each of the feet (x4) to make it easier to slide on to the bath board



Slide the foot along the rails and repeat so all four feet are correctly in place. Do not tighten up just yet as you will do this when you fit the bath board onto the bath

# How to Attach the Bath Board Handle



The optional bath board handle comes with three parts – a handle and two fixing plugs



Right side up, slot the handle in. The handle will only fit at one end of the board (same end as soap dish)



Before fitting, make sure the handle inclines inwards along the board. This ensures there is space for you to hold onto the handle



Turn the bath board over and tap the plugs in to each hole where the handle came through. No tools are required. The handle is now fitted

### How to Fit



Place the board over the top of the bath, opposite end to the taps. Make sure the soap dish (and handle, if supplied) are closest to the wall. Push down firmly so the suckers grip the bath rim



From above, with one hand, slide one foot out to make firm contact with the side of the bath. With the other hand, tighten the adjuster knob underneath



Before going any further, do a visual check that the internal bath width does not exceed the safety guide mark on both ends of the board before continuing



Repeat with all four feet. Try moving the board side to side - it should be fixed firmly in place. The bath board is now ready to use




Max. Safe User Weight 200kg (31 stone)

#### CHECK, EVERY TIME BEFORE USE, THE BOARD IS FIRMLY IN PLACE

It is not recommended that the bath board be fitted where there is an overlap of more than 25mm



For safety reasons never sit on the edge of the bath board if it overlaps the bath

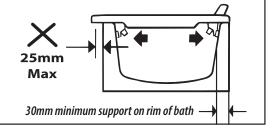

#### The bath/shower board will not fit a corner bath

## Please take care getting in and out of the bath



Sit on the end of the board with your hands either side



Lift each leg - one at a time - into the bath



Then slide across to the middle of the board

# Taking care of the **KINGFISHER**™ Bath Board

- **1 a)** We recommend to keep clean **between uses**, using non-abrasive disinfectant suitable for plastic baths, not forgetting the feet
  - b) <u>Before re-issue</u>, clean and disinfect product using standard detergents and disinfectant wipes Also compatible for automatic cleaning, using fluid with pH 5-8, up to 105°C for 10 minutes
- 2 Aways inspect the board for wear and tear and secure fitting before each use
- 3 Three year warranty, spare parts available

## This device is intended to be used by the following patient groups:

- Elderly or less mobile
  - No longer able to safely climb in/out of bath
  - Reduced mobility or reduced strength to transfer safely
  - Ability to sit unsupported/unaided
- Not suitable for
  - Users unable to sit without support
  - Children

## Disposal/End of use



- Community equipment premises product must be cleaned and disinfected prior to disposal within the local facilities waste management guidelines
- Home use product should be cleaned and disinfected prior to disposal in domestic waste

## **Storage**

- DO NOT store in sunlight
- Operating temperature +10°C / +40°C
- Storage temperatures -10°C / +40°C











#### The Helping Hand Company Ltd

Bromyard Road, Ledbury, Herefordshire HR8 1NS

Tel: **01531 635678** 

sales@**HelpingHand**.co.uk

**HH**ADL**Essentials**.com



INTERNATIONAL ASSOCIATES AUDITING & CERTIFICATION LTD

The Black Church, St. Mary's Place, Dublin, D07 P4AX Ireland







