

FOLDING COMMODE C017



Introduction

The folding commode has a robust yet lightweight steel frame with a 120kg (19 stone) weight capacity. It features an easy to use folding mechanism and plastic seat, armrests and backrest. It comes with a 5 litre potty and lid.

Users with visual, reading or cognitive disabilities should seek advice from a professional care provider for an appropriate format. If this is not viable, users should contact their provider or Drive DeVilbiss Healthcare Ltd.

Intended Environment

Domestic or residential setting where aid to user is required.

Intended Users

Any mobility restricted user who requires assistance.

Assembly

Holding the back of the commode, pull down on the front of the seat. The front and rear of the frames will separate. Once folded out, secure the frame by locking the plastic catches on both sides of the commode (see photo below)



Slide the potty (and lid if required) underneath the seat from behind (see photo below), ensuring that the potty is securely placed on the two rails underneath the seat. Ensure the handle is on the left or right side and at a 45° angle to aid insertion.



Folding the commode

Remove the potty by pulling it out from underneath the seat.

Unlock both catches and pull up on the front of the seat whilst holding the back.

Safety

The maximum weight capacity of the commode is 120kg (19 stone).

For the best stability, try to ensure the user's weight is distributed as centrally and evenly as possible across the commode.

Operation

The commode has two sturdy armrests to aid users when sitting or getting up from the commode.

Please see previous sections for details on potty removal and insertion.

The potty comes with a lid, to be used when transporting the potty.

Cleaning

The commode should be wiped clean with hot water and a mild detergent or disinfectant. The bucket and lid can be removed for cleaning. The plastic parts of the commode may also be cleaned with a hosepipe, or similar equipment, if required.

Technical Specification

Unfolded dimensions (D x W x H): 490 x 540 x 770mm

Folded dimensions

(exc. potty) (D x W x H): 150 x 540 x 780mm

Seat dimensions (D x W x H): 380 x 400 x 450mm

Weight: 5kg

Potty capacity: 5 litres

Weight capacity: 120kg (19 stone)

Warranty Details

Your Drive DeVilbiss branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive DeVilbiss stockist.

This does not affect your statutory rights.

Contact Details

Drive DeVilbiss Healthcare Ltd

Sidhil Business Park, Holmfield, Halifax,
West Yorkshire, HX2 9TN, Great Britain

info@drivedevilbiss.co.uk
www.drivedevilbiss.co.uk

Tel: +44 (0) 845 0600 333

Drive DeVilbiss Sidhil Ltd

4 Trench Road, Mallusk, Newtownabbey,
BT36 4TY, Northern Ireland