REF	Description
091080027	Standard
091080068	Width Adjustable
091080076	Deluxe Version

## Intended Uses

- To help increase independence when using the toilet, particularly by compensating for impaired movement.

## Assembly and Use

### Standard Homecraft Stirling Toilet Frame (091080027).

This toilet frame comes with the fixed-width frame assembled. For height adjustment, see **Adjusting the height**. For final positioning, see **Positioning the seat and frame**.

### Width-Adjustable Homecraft Stirling Toilet Frame (091080068) and Width Adjustable Toilet Frame Stirling With Padded Arms (091080076).

- The frame is shipped with the seat (labelled **D** in Figure 1) attached to the cross tubes (**B**). Remove the seat from the cross tubes by pulling at the front and back.
- Use a thumb and forefinger to pry the pin-clips (**C**) from the cross tubes.
- Lay one of the side frames (**A**) on the floor and position one of the cross tubes over the extending rod.
- Align the holes in the two parts and secure with a pin-clip. For a standard width, use the holes in the extending tubes that are closer to the frame; for a wider frame, use the holes that are farther from the frame.
- Repeat with the second cross bar.
- Lay the second frame on the floor and put the first frame assembly onto it, lining up the cross tubes and the extending rods. Turn the frame so the feet are up and secure the cross tubes.
- Stand the frame on the floor and position the seat in the middle of the cross tubes. Push it firmly into place.

## Adjusting the height

- With the frame lying on one side, remove the pin-clips (**F**) and slide the leg sleeves (**E**) to the desired height. Be sure all of the legs are adjusted to the same height and replace the pin-clips.

## Positioning the seat and frame

- Raise the toilet seat that is attached to the toilet. The Homecraft Stirling Toilet Seat should be positioned directly over the toilet bowl (see Figure 2).

## Care and Cleaning

All parts can be wiped using a brush or cloth and a solution of mild detergent or household disinfectant.

## Cautions

- 091080027, 091080068 and 091080076: The toilet seat is not to be used to support more than 190 kg / 420 lb. / 30 st.
- Before use, insure the toilet seat and frame are securely in place before putting weight on them.

Serious events experienced from using this product should be reported immediately to Performance Health and to the local Competent Authority.

Symbols Glossary located at <http://www.performancehealth.com/medical-device-glossary>.

Please retain these instructions for future reference.

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  
**Performance Health International Ltd.**  
 Nunn Brook Road  
 Huthwaite, Sutton-in-Ashfield  
 Nottinghamshire, NG17 2HU, UK  
 Tel: +44 1623 448 706  
[www.performancehealth.co.uk](http://www.performancehealth.co.uk)

  
 Performance Health France  
 13 rue André Pingat CS 10045  
 51 724 Reims Cedex France

**Distributed in the United States By:**  
 Performance Health Supply, LLC  
 W68 N158 Evergreen Blvd  
 Cedarburg, WI 53012, USA  
 +1800-323-5547  
[www.performancehealth.com](http://www.performancehealth.com)

**Distributed in Australia By:**  
 Ausmedic Australia, Trading as  
 Performance Health ANZ  
 Unit 3, 3 Basalt Road  
 Pemulwuy NSW, 2145, Australia  
 1300 473 422  
[www.performhealth.com.au](http://www.performhealth.com.au)

**Distribué en France par:**  
 Performance Health France  
 13 rue André Pingat CS 10045  
 51 724 Reims Cedex France  
 00(33) 03 10 00 79 30  
[www.performancehealth.fr](http://www.performancehealth.fr)

**EC REP**  
**CEpartner4U**  
 Esdoornlaan 13  
 3951 DB Maarn  
 The Netherlands  
[www.cepartner4u.com](http://www.cepartner4u.com)