

Lifemax

FAR Infrared Heated Lap Blanket

SKU: 1359



WWW.LIFEMAXUK.CO.UK


Thank you for choosing the Lifemax Far Infrared Heated Lap Blanket – your own portable hug!

To get the most out of your purchase, please thoroughly read these instructions.

Operating Instructions

With minimal power consumption, your blanket offers all the benefits of Far Infrared heat and can be used anywhere there is a wall or lighter socket.

For **use with a wall socket** connect the supplied mains adapter to the power lead running from the blanket, as shown below. Plug the mains adapter into a wall socket.

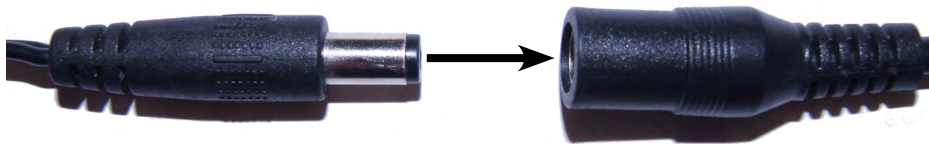

For **use with a lighter socket** connect the supplied car adapter to the power lead running from the blanket, as shown above. Plug the car adapter into a lighter socket. Ensure that the position of the power lead does not impede the operation of your vehicle.

Always unplug the mains or car adapter before connecting or disconnecting from the power lead.

The heating element in your blanket is centrally located, allowing it to be used as a lap blanket or worn as a heated shawl.

Do not sit or place heavy objects on top of your lap blanket, doing so can stretch the heating elements causing them to be compromised.

For **use as a lap blanket**, place the blanket horizontally over your knees, such that an equal amount of material falls either side of your lap. Using the integrated temperature control, select the heat setting that is most comfortable for you, as illustrated below.



For **use as a heated shawl**, place the blanket centrally over the shoulders with the power lead at the bottom of the blanket. Draw the shawl around the shoulders and affix the top corners using the button and loop provided, as illustrated.

Using the integrated temperature control select the heat setting that is most comfortable for you, as illustrated previously.

Everyone has their own tolerance to heat, but our **recommended period of use** is one hour – this will also prolong the life of your blanket. For your safety and comfort, the unit will automatically switch off after a period of one hour. This period can be reset by reselecting the heat setting. To **switch the unit** OFF with the on/off button image from the controller

Cleaning, Storage and Safety


To **clean your blanket**, spot clean using mild soap and water or upholstery cleaner. Always unplug your blanket before cleaning and never immerse in water or machine wash.

Your blanket may be folded or rolled for **storage** and should be kept in a cool, dry location. Take care not to damage the power lead and connector during storage.

To minimise the risk of injury to the user or damage to your blanket, follow these **safety instructions**.

1. Use the blanket with adult supervision and never leave unattended when in use. Those with restricted mobility such as young children or the disabled may be injured if they cannot remove the blanket.
2. Never remove or pierce the blanket cover. The use of pins or metallic fasteners represents a serious risk of electric shock.
3. Do not place heavy objects on the blanket or operate when folded or if the power lead or cover becomes damaged.
4. Do not use in areas where aerosol products are being used or oxygen is being administered.
5. If the unit becomes wet or immersed in water, do not attempt to retrieve or operate. We do not recommend the product is used outside.

6. **Do not use this product in the treatment of undiagnosed conditions, for example undiagnosed abdominal pain. Avoid using the blanket on swollen or inflamed areas, where the skin is lacerated or where the user suffers from Varicose Veins.**
7. **Those with sensitive skin or diabetes should seek medical advice before using this product.**
8. **This product is not recommended for use during pregnancy.**
9. **Those with a pacemaker should consult their GP prior to use.**

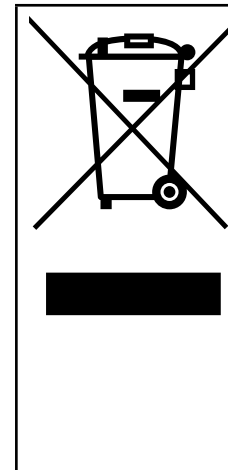

Protect our environment: do not dispose of electrical equipment in your domestic waste. Please return any electrical equipment that you will no longer use to the collection points provided for their disposal. These simple steps can help to avoid the potential effects that incorrect waste disposal can cause on the environment and human health. This will contribute to recycling and other forms of re-use of electrical and electronic equipment. Information concerning where the equipment can be disposed of can be obtained from your local authorities.

Please retain this information for future reference.



Made in China

Lifemax Limited, 1 Pipers Court,
Thatcham, Berkshire, RG19 4ER

Lifemax (Ireland), 51 Bracken Road,
Sandyford, Dublin, D18 CV48

www.lifemaxuk.co.uk



WWW.LIFEMAXUK.CO.UK

01635 874323