

Lifemax


Heated Back and Seat Massager

SKU: 226R

Thank you for choosing the Lifemax Heated Back and Seat Massager.

Your new massager delivers soothing heat and a gentle massage to the upper and lower back, hips and thighs. The five massage motors offer pulse, tapping, rolling, auto and kneading modes, as well as four levels of intensity to relieve your aches and pains.

Relax secure in the knowledge that your massager uses superior safety technology. Made using advanced fire retardant materials of the highest quality to meet the Fire and Furnishings (Fire)(Safety) Regulations.

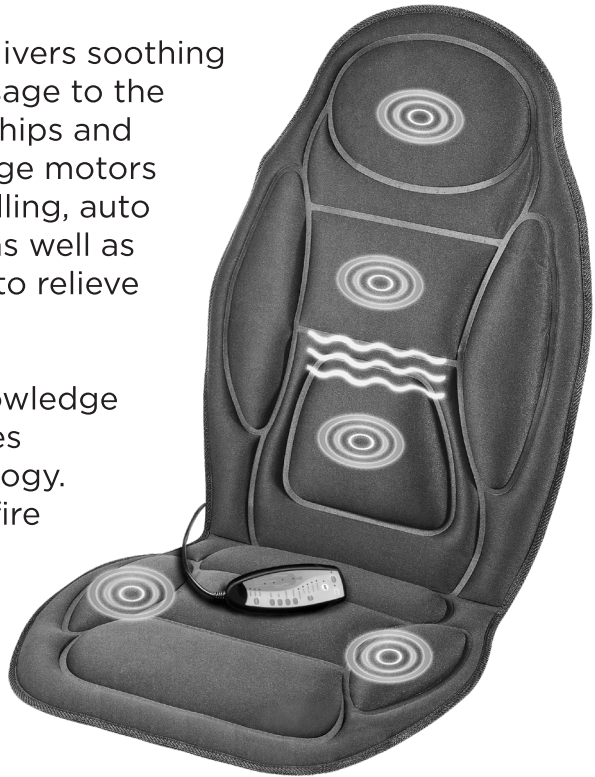

To get the most out of your purchase, please thoroughly read these instructions.



To minimise the risk of injury to the user or damage to your massager, follow these few safety instructions.

1. Use with adult supervision and never leave unattended when in use. Those with restricted mobility such as young children or the disabled may be injured if they cannot switch off the massager.
2. Never remove or pierce the cover and refrain from opening the zipper. The use of pins or metallic fasteners represents a serious risk of electric shock.
3. Do not operate the massager when folded or if the power lead or cover becomes damaged.
4. Do not use in areas where aerosol products are being used or oxygen is being administered.
5. If the unit become wet or immersed in water, do not attempt to retrieve or operate. We do not recommend the product is used outside.
- 6. Do not use this product in the treatment of undiagnosed conditions, for example undiagnosed abdominal pain. Avoid using on swollen or inflamed areas, or where the skin is lacerated.**
- 7. Those with sensitive skin or diabetes should seek medical advice before using this product.**
- 8. This product is not recommended for use during pregnancy.**
- 9. Those with a pacemaker or any health concerns should consult their GP prior to use.**
10. Before switching on, ensure the electrical voltage and frequency of the circuit correspond to the adapter.
11. Disconnect from any power source before moving or cleaning.





Operating Instructions

Your massager can be used anywhere there is a wall or lighter socket and incorporates a unique strapping system, which allows you to fasten it to almost any chair or seat. Although the cover material is colourfast, we recommend placing a towel or cover under the unit during initial usage or when used on light colour furniture.

Set the cushion in your seat and fasten the elastic straps firmly around the seat back. Once in place, your massager will not slip or slide around.

For **use with a wall socket** connect the supplied mains adapter to the power lead running from the massager, as



shown below. Plug the mains adapter into a 230 volt AC wall socket.

For **use with a lighter socket** connect the supplied car adapter to the power lead running from the massager, as shown above. Plug the car adapter into a 12 volt lighter socket. Ensure that the position of the power lead does not impede the operation of your vehicle.

Always unplug the mains or car adapter before connecting or disconnecting from the power lead.



Understanding your Remote Control

The **PWR** button turns the massager on and off

The **INT** button adjusts the strength of your massage. Choose from 4 levels of intensity.

The **HEAT** button turns the lumbar heat on and off

The **MODE** button chooses from 5 different massage modes:
Pulsating
Tapping
Rolling
Kneading
or Auto.

Auto cycles between modes.

Customise your massage area by choosing to turn on or off for:

UPPER BACK
LOWER BACK
HIPS
THIGHS

The **TIMER** allows you to choose between 15, 30 and 60 minute programs.



Everyone has their own tolerance to heat so we recommended choosing the 15 minute program initially - this will also prolong the life of your massager

For your safety and comfort, the unit will automatically switch off after a maximum of 60 minutes. Prolonged continuous use of your massager can cause excessive heating and shorten its working life.



Cleaning, Storage and Safety

To clean your massager, spot clean using mild soap and water or upholstery cleaner. Always unplug your massager before cleaning and never immerse in water or machine wash.

Your massager may be folded for storage and should be kept in a cool, dry location. Take care not to damage the power lead and connector during storage.

Please retain this information for future reference.



Lifemax Limited, 1 Pipers Court,
Thatcham, Berkshire, RG19 4ER

Lifemax (Ireland), 51 Bracken Road,
Sandyford, Dublin, D18 CV48

www.lifemaxuk.co.uk



Made in China

Protect our environment: do not dispose of electrical equipment in your domestic waste. Please return any electrical equipment that you will no longer use to the collection points provided for their disposal. These simple steps can help to avoid the potential effects that incorrect waste disposal can cause on the environment and human health. This will contribute to recycling and other forms of re-use of electrical and electronic equipment. Information concerning where the equipment can be disposed of can be obtained from your local authorities.

