THUASNE

Ortel C1 Classic

fr	Collier cervical souple.....	6
en	Soft cervical collar.....	7
de	Weiche Halskrause.....	8
nl	Flexibele nekbrace.....	9
it	Collare cervicale morbido.....	11
es	Collarín cervical flexible.....	12
pt	Colar cervical flexível.....	13
da	Smidig nakkekrave.....	14
fi	Taipuisa kaulatuki.....	16
sv	Mjuk nackkrage.....	17
el	Αυχενικό κολλάρο μαλακό.....	18
cs	Pružný krční límec.....	19
pl	Kołnierz elastyczny do unieruchomienia szyi.....	21
lv	Mīksta kakla ortoze.....	22
lt	Lankstus kaklo įtvaras.....	23
et	Pehme emakakaela krae.....	24
sl	Opornica za vrat, prožna.....	26
sk	Ohybný krčný golier.....	27
hu	Lágy nyakmerevítő.....	28
bg	Мека цервикална яка.....	29
ro	Guler cervical flexibil.....	31
ru	Мягкий шейный воротник.....	32
hr	Mekana ortoza za vratnu kralježnicu.....	33
zh	软质颈圈.....	35
ar	طوق للعنق مرن.....	36




	a	b		
0	24 - 29 cm	6 cm	9 cm	11 cm
1	28 - 33 cm			
2	34 - 39 cm			
3	40 - 46 cm	-		

fr	(a) Tour de cou (b) Hauteur de cou	lv	(a) Kakla apkārtmērs (b) Kakla augstums
en	(a) Neck circumference (b) Neck height	it	(a) Kaklo apimtis (a) Kaklo aukštis
de	(a) Halsumfang (b) Halshöhe	et	(a) Kaela ümbermõõt (b) Kaela pikkus
nl	(a) Omtrek van de nek (b) Hoogte van de nek	si	(a) Obseg vratu (b) Dolžina vratu
it	(a) Circonferenza del collo (b) Altezza del collo	sk	(a) Obvod krku (b) Výška krku
es	(a) Circunferencia del cuello (b) Altura del cuello	hu	(a) Nyak kerülete (b) Nyak magassága
pt	(a) Perímetro do pescoço (b) Altura do pescoço	bg	(a) Размер на яката (b) Височина на яката
da	(a) Halsens omkreds (b) Halsens højde	ro	(a) Circumferința gâtului (b) Înălțimea gâtului
fi	(a) Kaulanpäärys (b) Kaulan korkeus	ru	(a) Обхват шеи (b) Высота шеи
sv	(a) Runt halsen (b) I höjd med halsen	hr	(a) Opseg vrata (b) Visina vrata
el	(a) Περιμετρος λαιμού (b) Ύψος λαιμού	zh	(a) 颈围 (b) 颈高
cs	(a) Obvod krku (b) Výška krku	ar	محيط العنق (a) ارتفاع العنق (b)
pl	(a) Obwód szyi (a) Wysokość szyi		

fr	Maintien	Effet proprioceptif
en	Support	Proprioceptive effect
de	Halt	Propriozeptive Wirkung
nl	Fixatie	Proprioceptieve werking
it	Sostegno	Effetto proprioiettivo
es	Sujeción	Efecto propioceptivo
pt	Manutenção	Efeito proprioceptivo
da	Støtte	Proprioceptisk effekt
fi	Tuki	Liikeaistiin kohdistuva vaikutus
sv	Stöd	Proprioceptiv effekt
el	Συγκράτηση	Αποτέλεσμα ιδιοδεκτικότητας
cs	Stabilizace	Proprioceptivní účinek
pl	Podtrzymywanie	Propriocepcja
lv	Atbalsts	Proprioceptīvs efekts
lt	Palaikymas	Propriocepčinis poveikis
et	Hooldus	Proprioceptiivne toime
sl	Oprijem	Proprioceptivni učinek
sk	Podpora	Proprioceptívny účinok
hu	Támasz	Proprioceptív hatás
bg	Поддръжка	Проприоцептивен ефект
ro	Sustinere	Efect proprioceptiv
ru	Стабилизация	Проприоцептивный эффект
hr	Podpora	Učinak proprioceptivnog vježbanja
zh	穩定	本体感
ar	الدعم	مفعول استقبال الحس العميق

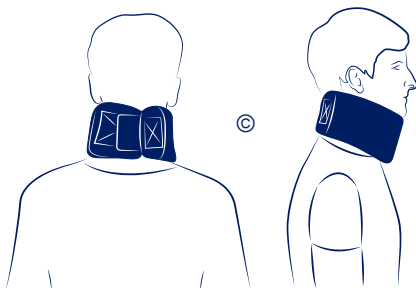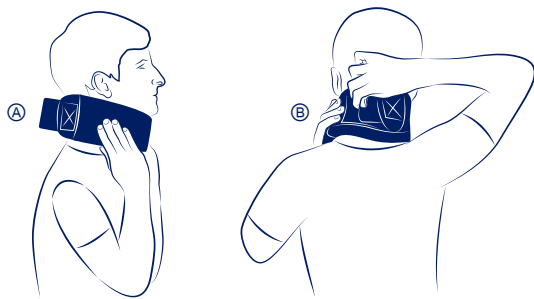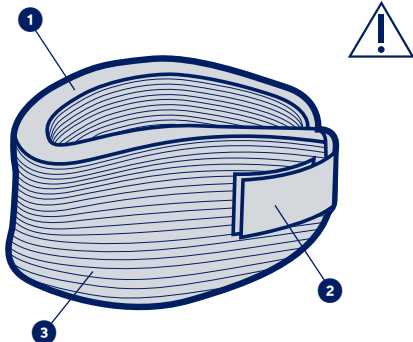MD

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SOFT CERVICAL COLLAR

Description/Destination

Rounded edges for appropriate comfort. ❶

Precise self-fastening adjustment. ❷

Convenient removable cover. ❸

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Cotton - polyamide - polyester - polyurethane.

Properties/Mode of action

Support.

Proprioceptive effect.

Indications

Cervical arthritis.

Cervical syndrome.

Dropped head syndrome.

Mild whiplash/mild sprain.

Post-operative use.

Rehabilitation after wearing a rigid cervical collar.

Torticollis (stiff neck).

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of traumatic injuries: severe sprain, fracture.

Do not use for prolonged wear without medical supervision.

Do not use the product if the diagnosis has not been confirmed.

Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

Choose the appropriate size to fit the patient, referring to the size chart.

Verify the product's integrity before every use.

It is recommended that a healthcare professional supervises the first application. For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device if it is damaged.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without excessive compression.

Do not wear the product in a medical imaging machine.

Do not wear the product while sleeping.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

In the event of long-term wear of the product, possible secondary risks related to muscle atrophy.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Open the self-fastener.

Place the collar around the neck ⊕.

Close the self-fastener at the nape of the neck ⊕.

Ensure the strap is not fastened too tight ⊕.

Care

See product label. Close the self-fastening tabs before washing. Hand wash.

Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2028902 (2022-03)