Sedens 410

User's Manual

Care for a Healthy Life

1. Intended Use

This Apex Hybrid Seat Cushion is an adjustable, air-and-foam filled, cellular-design wheelchair support surface that intended to provide continuity of pressure care from bed to chair. The cushion is compatible with all pumps that use 1-in-2 Alternating Apex quick connectors (except Auto systems) to have alternating functions. There is no weight limit, yet the cushion must be properly sized to the individual. It also provides following purposes: to help and reduce the incidence of pressure ulcers while optimizing patient comfort.

NOTE, CAUTION AND WARNING STATEMENTS

NOTE: Indicates some tips.

CAUTION: Indicates correct operating or maintenance procedures in order to prevent damage to or destruction of the equipment or other property

WARNING: Calls attention to a potential danger that requires correct procedures or practices in order to prevent personal injury.


SYMBOLS

	Refer to instruction manual/ booklet		Humidity Limitation
	Do Not Dry Clean		Do Not Iron
	Tumble Dry, Normal, Low Heat		Machine wash, regular/ normal, 95 °C [203 °F]
	Do Not Bleach		Temperature Limitation
	Manufacturer		Authorised representative in the European community

2. Cushion Description

1. Stretch Cover
2. Zipper
3. Air cells with foam inside
4. Quick Connector (to Apex pump)


NOTE: Please refer to the pump manual for the respective pump description.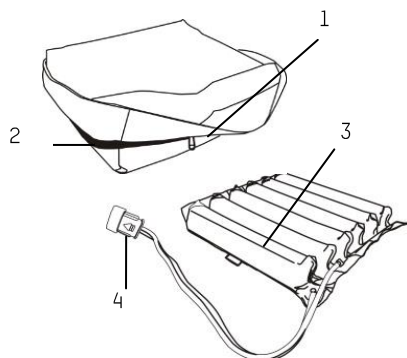

3. Installation & Adjustment

Alternating Hybrid Cushion Use- With Apex Pump

NOTE: The Sedens 410 Hybrid Cushion is compatible with all pumps that use 1-in-2 Apex Quick Connectors, except Auto systems.

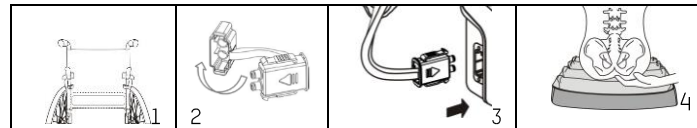1. Place Sedens 410 hybrid cushion in wheelchair seat so the user will be properly supported and immersed when in their normal sitting position.
2. Uncapped the quick connector and connect to a compatible Apex pump. A "click" sound will be heard when the connection is completed.
3. Install the pump according to the pump manual.
4. Have the user sit on the cushion. Check to make sure the cushion is properly sized to the user and wheelchair.
5. Adjust an appropriate pressure setting to ensure the patient's comfort without bottoming out.
6. Keep hand under user's lowest bony prominence and make sure there is between 1/2 in. (1.3cm) to 1 inch (2.5 cm) of cushion between the user's lowest bony prominence and the seating surface. If not enough space in between, please adjust the pump pressure to higher level.

4. Cleaning

Cover Material : Stretch	
--------------------------	--

Wipe down the mattress unit with a damp cloth pre-soaked with warm water containing a mild detergent, or chlorine bleach followed by an approved intermediate level disinfectant. Top cover can also be completely removed for laundry with indicated temperature; however, it is recommended that the user still check with local policy to determine the time/ temperature ratio required achieving thermal disinfection. After cleaning, please avoid direct sunlight or dust areas. All parts should be air dried thoroughly before use.

5. Technical Specifications

Cushion	Specification
Dimension (L x W x H)	44 X 41 X 9 cm or 17.3" x 16.1" x 3.5"
Weight	1.1 kg or 2.4lb
Max. Weight Capacity	There is no weight limit, yet the cushion must be properly sized to the individual

NOTE:

1. Pump specification/EMC information please refer to the pump manual.
2. Consult the distributor for other technical documents.
3. Please follow national requirements to dispose of the unit properly.
4. The manufacture reserves the right to change product specification without prior notice.

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Print-2018/All rights reserved
Apex Medical Corp.
Apex Medical (Kunshan) Corp.